FASTest® CRYPTO Strip

ad us. vet.

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FASTest® CRYPTO Strip

ad us.vet.

Test-kit for the detection of
Cryptosporidium parvum in calf feces

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3. Test Kit Components

1 Test-kit **FASTest® CRYPTO** Strip:


- 2 or 10 Dipstick Tests coated with monoclonal antibodies.
- 2 or 10 Sample Tubes with 3,0 ml buffer diluent.
- 1 Instructions for use.

4. Storage and Shelf Life

- Store at room temperature (+15°C - +25°C).
- Stored correctly the product can be kept up to the expiry date.
- Avoid the Test kit being subjected to excess heat or freezing.

1. SPECIMEN COLLECTION AND DILUTION

- Open the sample tube and take out the sample spoon, provided with the kit.
- Take a little portion (0,4g) of feces with the spoon and put it back into the sample tube. Screw on the cap tightly avoiding leakage.(fig.1)
- If possible let the test material dissolve in the sample tube.
- Shake the tube until the sample has been dissolved into the buffer diluent.(fig.2)
- Leave the test tube until the large particles have settled to the bottom of the tube.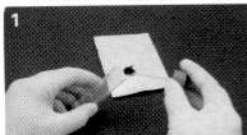

1. Introduction

Diarrhea in calves during the first two weeks of life is mostly caused by infectious agents like Rotavirus, Coronavirus, E. coli spp and more frequently Cryptosporidium parvum. Finding a quick diagnosis on-site is often hard, an mostly based only on clinical signs. Facing the problem, that most of the diarrhea causing agents did not show a pathogenic clinical picture, specially in cases of mixed infection, it is necessary to have a diagnostic tool to detect the diarrhea causing agent.

Using **FASTest® CRYPTO** Strip allows the veterinarian, the rapid specific diagnosis and therapy, e.g halufuginone, in case of a present cryptosporidiosis. The identification of animals with cryptosporidiosis saves time and can be used for developing optimal strategies in conjunction with the farmer to optimise calves health and farm productivity.

Concerning the cross-transmission from calves to humans and the high disease potential in immunocompromised humans, it is important to know about dissemination of Cryptosporidium parvum in the livestock.

5. Information on the Test Sample Material

Only feces can be examined using the **FASTest® CRYPTO** Strip. The test sample material can be stored for up to 14 days at 2°-8°C.

6. Special Information


- FOR VETERINARY USE ONLY!
- Do not use the test once the expiry date has passed.
- Do not use reagents from different kits.
- The sample material must be seen as potentially infectious.
- Follow instructions for use precisely.

2. TEST PROCEDURE

- Remove the dipstick from its pouch by tearing along the split.
- Label the dipstick with a patient name or ID number.
- Introduce the dipstick vertically with the arrows pointing downwards into the sample tube with the feces-buffer-fluid.(fig.3)
- Remove the dipstick from the sample tube with the feces-buffer-fluid after 1 Minute or longer.(fig.4)
- Place the dipstick on a flat surface.

3. INTERPRETATION OF TEST RESULTS

Read the results 5 minutes after the dipstick has been placed in the sample tube with feces-buffer-fluid. The test result will be stable over a long term.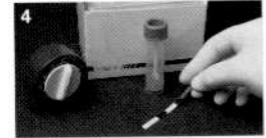

2. Test Principle

The **FASTest® CRYPTO** Strip contains two unique monoclonal antibodies. One of these monoclonal antibodies is bonded to Latex-particles, the second is immobilized in the Test zone of the strip. If the stool sample extract contains *Cryptosporidium parvum* antigens these will form an antigen-antibody complex with the Latex-particles. As this complex migrates along the TEST zone of the membrane to the immobilized capture antibody, a red line is formed indicating a positive test result. The remaining conjugate migrates to a second antibody on the CONTROL zone of the strip forming a second blue line. This indicates proper performance of the test. The colour intensity varies and is dependent on the quantity of antigens in the sample.

IMPORTANT INFORMATION

The detection of *C. parvum* using the **FASTest® CRYPTO** Strip is an important but only one diagnostic step to verify the Diagnosis „Cryptosporidiosis“.

The Interpretation of the test result should be taken using all available informations (case history, clinical features a.s.o.)! A positive test result does not rule out the presence of other diarrhea causing pathogens in the sample. A single negative **FASTest® CRYPTO** Strip does not rule out the presence of *C. parvum* antigens under the cut-off limit off 50-100 Oocytes/100 µg feces.

Also a negative **FASTest® CRYPTO** Strip does not rule out a peristent *C. parvum* infection, because of the intermittent shedding of *C. parvum* oocysts or their antigens. In case of a reasonable suspicion of an infection exists, another sample (if possible a pooled fecal sample collected over three days) **1 Week** later should be investigated to confirm the negative test result. Incomplete or no line colourations may be the result of using too much feces sample !

NOTE

Other band colourations and discolourations which appear after 10 minutes have no diagnostic value. These phenomena may be the result of using too much stool sample.

LIABILITY

The entire risk due to the performance of this product is assumed by the purchaser. The manufacturer shall not be liable for indirect, special or consequential damages of any kind resulting from use of this product.

POSITIVE TESTRESULT

In addition to the blue line in the CONTROL zone, a clearly visible red line also appears in the TEST zone. The feces sample contains *Cryptosporidia* spp.(fig.5)


NEGATIVE TESTRESULT

Only the blue line appears in the CONTROL zone. No visible line is seen in the TEST zone. The feces sample contains no *Cryptosporidia* spp.(fig.6)

INCONCLUSIVE TESTRESULT

If there are no coloured (red/blue) lines, neither in the TEST zone nor the CONTROL zone of the dipstick, visible after 10 minutes. The test should be repeated using a new dipstick.

POSITIVE TESTRESULT

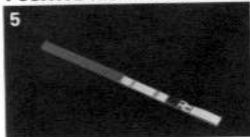NEGATIVE TESTRESULT