FASTest® E.coli-K99 Strip

ad us.vet.

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FASTest® E.coli-K99 Strip

ad us.vet.

Test-kit for the detection of E.coli
main pilar antigen K99 (F5) in calf feces

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3. Test-Kit Components

1 Test-kit **FASTest® E.coli K-99** Strip:

- 2 or 10 Dipstick Tests coated with monoclonal antibodies.
- 2 or 10 Sample Tubes with 3,0 ml buffer diluent.
- 1 Instructions for use.

4. Storage and Shelf Life

- Store at room temperature (+15°C - + 25°C).
- Stored correctly the product can be kept up to the expiry date.
- Avoid the Test kit being subjected to excess heat or freezing.

Please note the two light-green quality control lines, one in the TEST- and one in the CONTROL zone. If they are not visible (e.g. in case of contact with humidity), the dipstick is not usable !

1. SPECIMEN COLLECTION AND DILUTION

- a. Open the sample tube and take out the sample spoon, provided with the kit.
- b. Take a little portion (0,4g) of feces with the spoon and put it back into the sample tube. Screw on the cap tightly avoiding leakage. (fig.1)
- c. If possible let the test material dissolve in the sample tube.
- d. Shake the tube until the sample has been dissolved into the buffer diluent. (fig.2)
- e. Leave the test tube until the large particles have settled to the bottom of the tube.

1. Introduction

Diarrhea is a common complaint in calves, typically in the first 7 to 14 days. Often caused by more than one pathologic agent like Rotavirus, Coronavirus, Cryptosporidium spp., Salmonella spp., Giardia spp. and especially **E.coli spp. enteric colibacillosis** is caused by **enterotoxigenic** strains of **E. coli (ETEC: K 99 (F5), F 41)**. Finding a quick diagnosis on-site is often hard and mostly based only on clinical signs. Facing the problem, that most of the diarrhea causing agents did not show a clear pathogenic clinical picture, specially in cases of mixed infection, it is necessary to have a diagnostic tool to detect the diarrhea causing agent.

Using **FASTest® E.coli K-99** Strip allows the veterinarian on clinical grounds, the rapid and specific diagnosis and therapy, in case of a present enteric colibacillosis. The rapid identification saves time which can be used to suggest management strategies that will prevent further outbreaks of disease optimising calves health and farm productivity. Concerning the cross-transmission from calves to humans it is important to know about the incidence of E. coli spp. in the livestock, being able to take great care when handling diarrheic animals, and fecal samples to avoid contaminating yourself and others.

5. Information on the Test Sample Material

Only feces can be examined using the **FASTest® E.coli K-99** Strip. The test sample material can be stored for up to 14 days at 2°-8°C.

6. Special Information

- FOR VETERINARY USE ONLY!
- Do not use the test once the expiry date has passed.
- Do not use reagents from different kits.
- The sample material must be seen as potentially infectious.
- Follow instructions for use precisely.

2. TEST PROCEDURE

- a. Remove the dipstick from its pouch by tearing along the split.
- b. Label the dipstick with a patient name or ID number.
- c. Introduce the dipstick vertically with the arrows pointing downwards into the sample tube with the feces-buffer-fluid. (fig.3)
- d. Remove the dipstick from the sample tube with the feces-buffer-fluid after 1 Minute or longer. (fig.4)
- e. Place the dipstick on a flat surface.

3. INTERPRETATION OF TEST RESULTS

Read the results 5 minutes after the dipstick has been placed in the sample tube with feces-buffer-fluid. The test result will be stable over a long term.

2. Test Principle

The **FASTest® E.coli K-99** Strip contains two unique monoclonal antibodies. One of these monoclonal antibodies is bonded to colloidal gold particles, the second is immobilized in the test zone of the strip. If the stool sample extract contains K99 main pilar antigens these will form an antigen-antibody complex with the gold particles. As this complex migrates along the TEST zone of the membrane to the immobilized capture antibody, a pink-purple line is formed indicating a positive test result. The remaining conjugate migrates to a second antibody on the CONTROL zone of the strip forming a second pink-purple line. This indicates proper performance of the test.

LIABILITY

The entire risk due to the performance of this product is assumed by the purchaser. The manufacturer shall not be liable for indirect, special or consequential damages of any kind resulting from use of this product.

POSITIVE TESTRESULT

In addition to the pink-purple line in the CONTROL zone, a clearly visible pink line also appears in the TEST zone. The feces sample contains E.coli-K99 (F5). (fig.5)

NEGATIVE TESTRESULT

Only the pink-purple line appears in the CONTROL zone. No visible line is seen in the TEST zone. The feces sample contains no E.coli-K99 (F5). (fig.6)

INCONCLUSIVE TESTRESULT

If there are no pink-purple lines, neither in the TEST zone nor the CONTROL zone of the dipstick, visible after 10 minutes. The test should be repeated using a new dipstick.

POSITIVE TESTRESULT

NEGATIVE TESTRESULT